

Type 2 Diabetes deserves better.

Clinical evidence for MiniMed™ 780G
System in Patients with Type 2
Diabetes on MDI





International guidelines explicitly recommend AID for type 2 diabetes on MDI.¹

Automated insulin delivery should be offered for diabetes management to youth and adults on MDI with type 2 diabetes.

	Baseline HbA1c > 8.0%		Overall	
	Run-in (N=38)	Study (N=35)	Run-in (N=95)	Study (N=91)
TDD, U	81.2 ± 38.8	105.3 ± 58.0	77.4 ± 38.5	91.8 ± 49.3
Total basal, U	46.4 ± 20.5	57.5 ± 30.7	46.0 ± 21.9	50.1 ± 26.6
Total bolus, U	34.8 ± 25.0	47.8 ± 35.7	31.4 ± 25.1	41.7 ± 28.8
Auto correction, %TB	--	34.2 ± 19.7	--	34.4 ± 22.5

Figure 1. Glycaemia and insulin delivery during the run-in and study periods. HbA1c, continuous glucose monitoring derived metrics, and insulin delivery for the overall group and stratified by baseline HbA1c >8.0%. AHCL, advanced hybrid closed loop; HbA1c, glycosylated hemoglobin.

† N=88, ‡ N=34

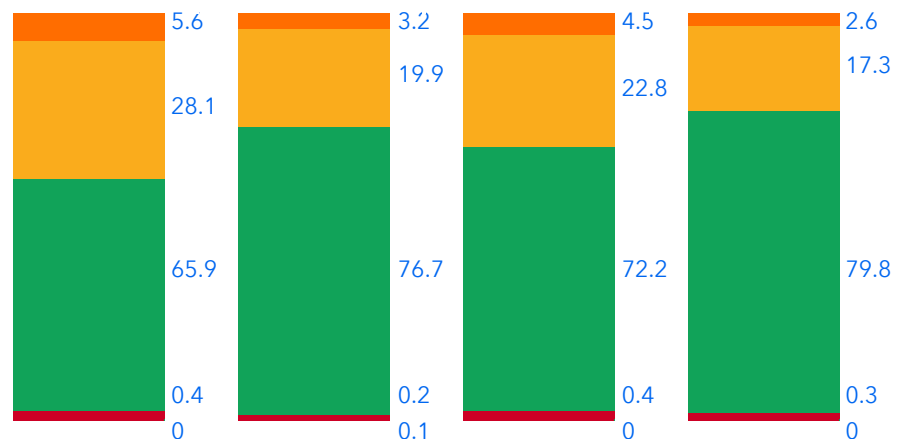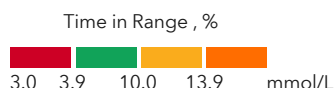

Clinically significant improvements in HbA1C, %TIR and %TAR.²

No increase in hypoglycaemia,
No significant change
in weight or BMI.²

7.2% HbA1c at 3 months.²

63% No prior pump or CGM experience.²


Don't wait to intensify.

Therapeutic inertia remains one of the greatest barriers to optimal outcomes in type 2 diabetes.¹



	Self reported T2D			
	All Settings		≥ 100 IU	
	All users (N=10,795)	TDD ≤ 100 IU (N=7,961)	TDD ≥ 100 IU (N=2,834)	Using recommended settings (N=868)
GMI, %	7.0 ± 0.5	6.9 ± 0.5	7.1 ± 0.5	7.2 ± 0.7 [†]
Recommended setting use, %	26.5	25	30.6	--
Mean SG, mmol/L	8.5 ± 1.3	8.4 ± 2.0	8.8 ± 1.4	--

Figure 2. Glycaemia outcomes of real world MiniMed™ 780G system users with self-reported T2D with or without a TDD of >100 IU. (TDD) Total Daily Dose of Insulin; (IU) Insulin Units; (T2D) Type 2 Diabetes.³

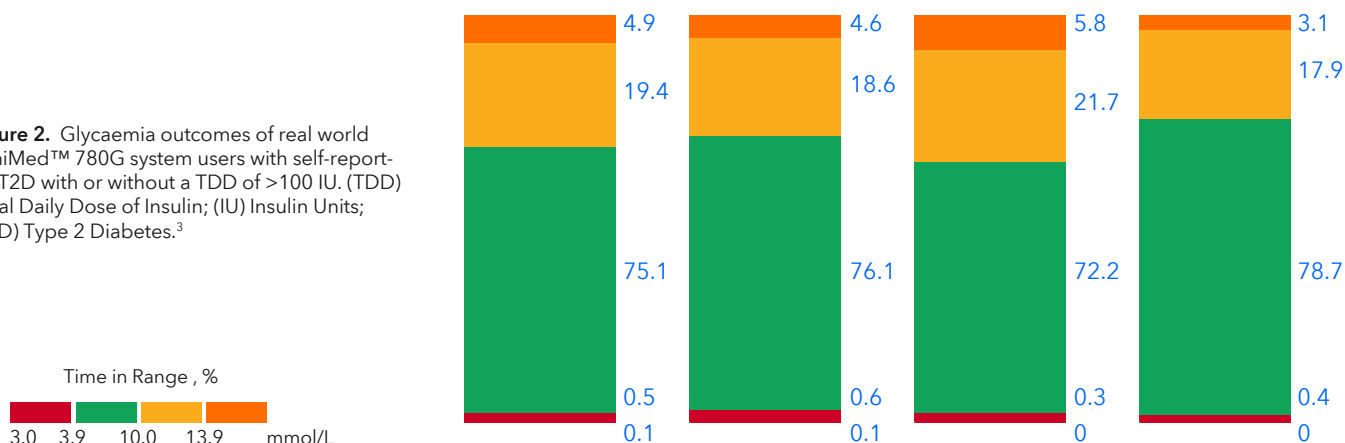

~65-70%

of insulin, was delivered automatically by the system.³

Not just a trial result.

Findings hold up across 26,427 real world users.³

Now indicated for adults with insulin-requiring type 2 diabetes.

Ask about our monthly plans, including sensors and consumables for system use.

MiniMed™ 780G System[^]



For Healthcare Professional Use Only. This material is intended solely for healthcare professionals and is not intended for distribution to, or use by, patients or the general public. Do not copy, reproduce, share, or redistribute this material without prior written authorization from MiniMed.

[^]Components sold separately.

[†]Users need to bolus on the system. The study showed user-initiated boluses (something participants were doing in their prior therapy) decreased.²

Please refer to the product insert or Instructions for Use for a complete list of indications, contraindications, warnings, precautions, and other important medical information.

For further information, contact your local MiniMed™ representative.

References 1. American Diabetes Association Professional Practice Committee, et al. Diabetes Care. 2026;49(Suppl 1):S150-S165. 2. Bhargava A, et al. Diabetes Technol Ther. 2025;27(5):366-375. 3. Thijs I, et al. J Diabetes Sci Technol. 2025;19322968251318373. Please refer to the product insert or Instructions for Use for a complete list of indications, contraindications, warnings, precautions, and other important medical information. For further information, contact your local MiniMed™ representative.

Medtronic Australasia Pty Ltd. Level 8, 11 Khartoum Road, Macquarie Park, NSW 2113 Australia. Tel: +61 2 9857 9000, Toll Free: 1800 777 808. Email: australia.diabetes@medtronic.com. Approval AU-DBA-2600010. ©2026 Medtronic MiniMed Inc. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc.™ Third-party brands are trademarks of their respective owners. All other brands are trademarks of a MiniMed company.